Invacare® I-Fit

9781E / 9780E

en	Shower Chair User Manual	2
de	Duschstuhl Gebrauchsanweisung	5
fr	Chaise de douche Manuel d'utilisation	8
it	Sedia da doccia Manuale d'uso	11
pt	Cadeira de duche Manual de utilização	14
nl	Douchestoel Gebruiksaanwijzing	17
no	Dusjstol Bruksanvisning	20

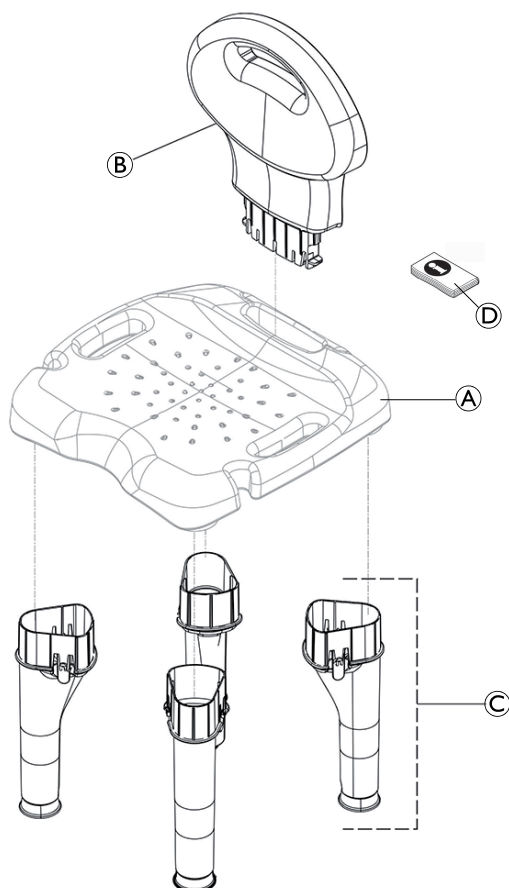
Fig. 1

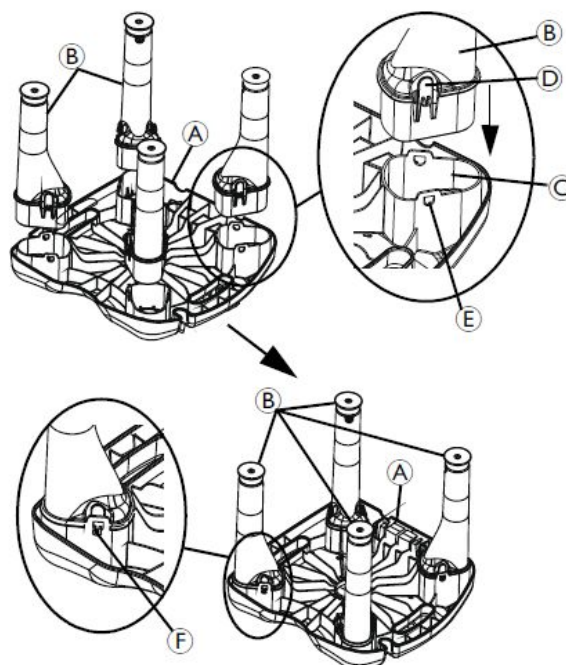
Fig. 2

This manual MUST be given to the user of the product.
BEFORE using this product, this manual MUST be read and saved for future reference.

Yes, you can.®

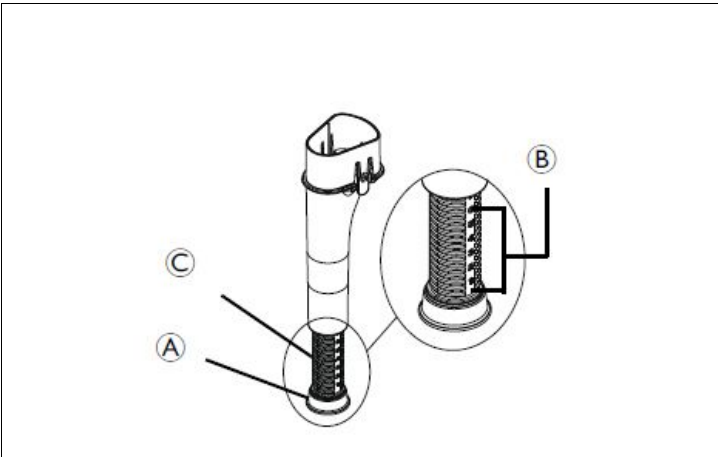
Fig. 3

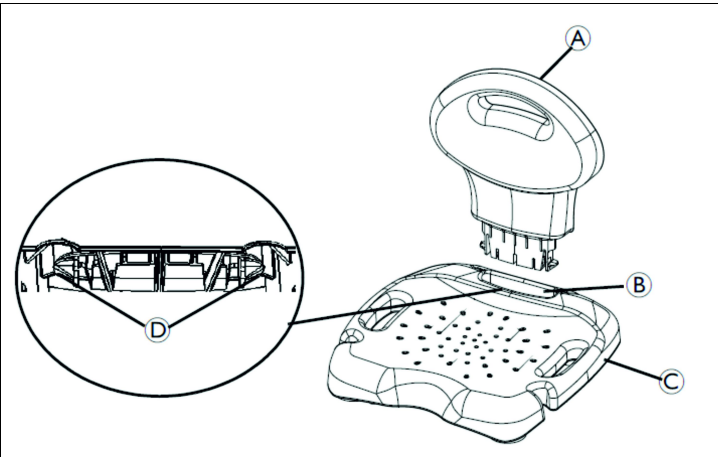
Fig. 4

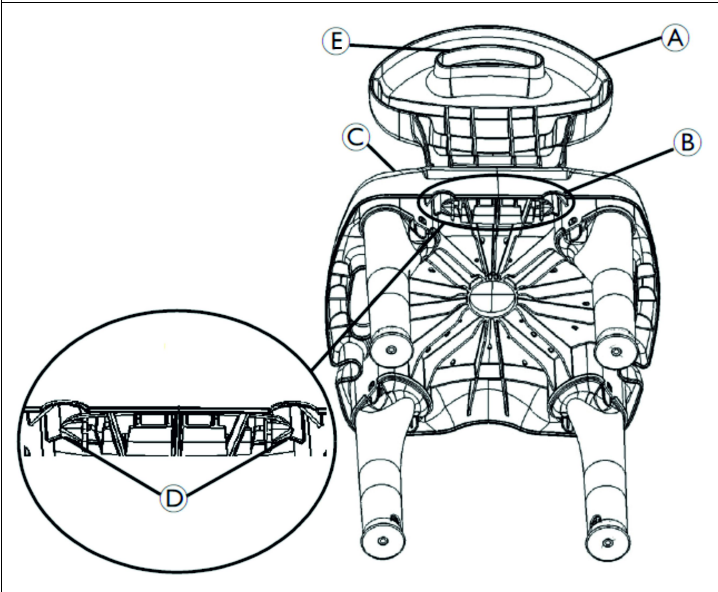
Fig. 5

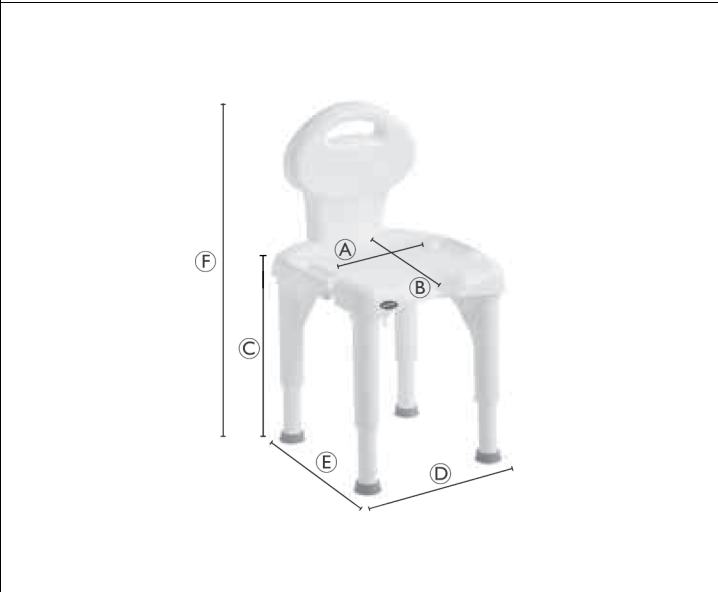
Fig. 6

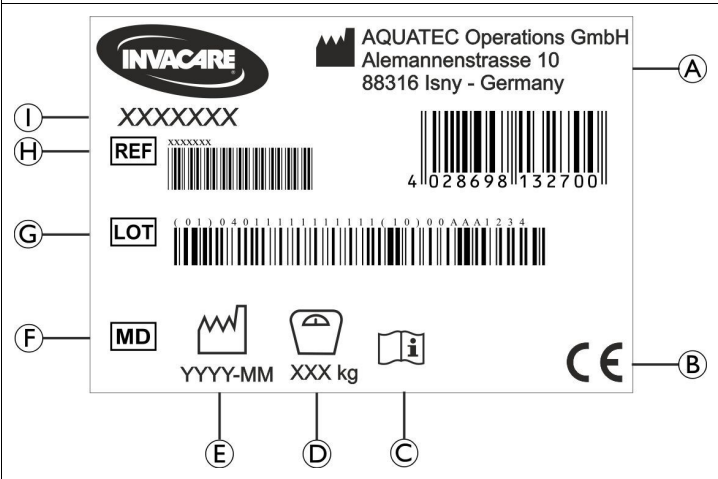
Fig. 7

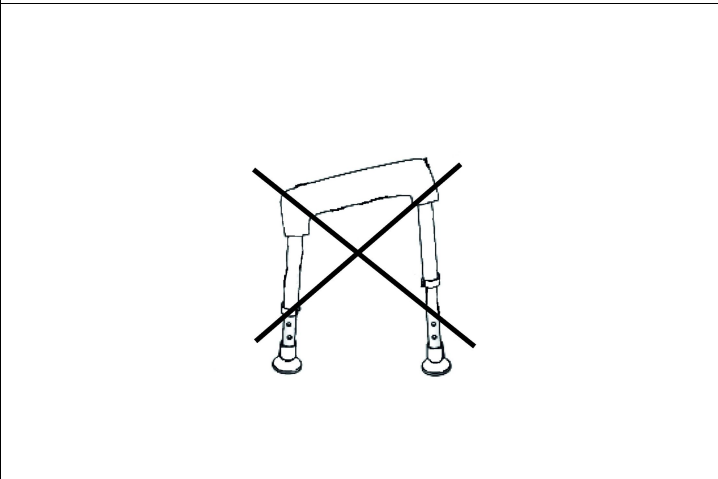
Fig. 8

en

1 Introduction

This user manual contains important information about the handling of the product. To ensure safety when using the product, read the user manual carefully and follow the safety instructions.

Invacare reserves the right to alter product specifications without further notice.

Before reading this document, make sure you have the latest version. You find the latest version as a PDF on the Invacare website.

If you find that the font size in the printed document is difficult to read, you can download the PDF version from the website. The PDF can then be scaled on screen to a font size that is more comfortable for you.

In case of a serious incident with the product, you should inform the manufacturer and the competent authority in your country.

2 Warranty Information

We provide a manufacturer's warranty for the product in accordance with our General Terms and Conditions of Business in the respective countries.

Warranty claims can only be made through the provider from whom the product was obtained.

3 Compliance

Quality is fundamental to the company's operation, working within the disciplines of ISO 13485.

This product features the CE mark, in compliance with the Medical Device Regulation 2017/745 Class I. The launch date of this product is stated in the CE declaration of conformity.

We are continuously working towards ensuring that the company's impact on the environment, locally and globally, is reduced to a minimum.

We only use REACH compliant materials and components.

3.1 Product-specific Standards

The product has been tested and conforms to EN 12182 (Assistive products for persons with disability) and related standards. For further information about local standards and regulations, contact your local Invacare representative. See addresses at the end of this document.

4 Service Life

The expected service life of this product is three years when used daily and in accordance with the safety instructions, maintenance intervals and correct use, stated in this manual. The effective service life can vary according to frequency and intensity of use.

5 Limitation of Liability

Invacare accepts no liability for damage arising from:

- Non-compliance with the user manual
- Incorrect use
- Natural wear and tear
- Incorrect assembly or set-up by the purchaser or a third party
- Technical modifications
- Unauthorised modifications and/or use of unsuitable spare parts

6 Safety

6.1 Safety Information

WARNING!

Risk of serious Injury or Damage

Improper use of this product may cause injury or damage.

- If you are unable to understand the warnings, cautions or instructions, contact a health care professional or provider before attempting to use this equipment.
- Do not use this product or any available optional equipment without first completely reading and understanding these instructions and any additional instructional material such as user manual, service manual or instruction sheet supplied with this product or optional equipment.

WARNING!

Risk of Falling!

- Do not use if defective.
- Do not make any unauthorised alterations or modifications to the product.
- Do not move or displace the product under load or during use.
- Do not tilt.
- Do not stand on this product.
- Do not use as a climbing aid.
- Do not use or store the product outdoors.
- All four legs must be adjusted to the same height.
- All four leg tips must be in contact with shower / tub floor at all times and on an even surface.

WARNING!

- Before first use, clean the product thoroughly.
- Before every use, check the product for damage and ensure that it is securely assembled.

CAUTION!

Risk of Injury

Non-original or wrong parts may affect the function and safety of the product.

- Only use original parts for the product in use.
- Due to regional differences, refer to your local Invacare catalogue or website for available options or contact your Invacare distributor. See addresses at the end of this document.

6.2 Labels and Symbols on the Product (Fig. 7)

Ⓐ	Manufacturer	Ⓕ	Medical Device
Ⓑ	European Conformity	Ⓖ	Lot number
Ⓒ	Read user manual	Ⓗ	Reference number
Ⓓ	Maximum user weight	Ⓘ	Product designation
Ⓔ	Production date		

The identification label is attached to the underside of the seat plate.

7 Scope of Delivery (Fig. 1)

- Ⓐ Seat plate
- Ⓑ Backrest (only for 9781E)
- Ⓒ Legs (4x)
- Ⓓ User manual

8 Intended Use

This product is intended to be used as a shower aid.

Users are any adults and adolescents with restricted or limited mobility (e.g.: legs and/or hips). If used without an assistant, the users must be able to keep their own balance.

Indication

Restricted or limited mobility (e.g.: legs and/or hips) or having balance issues to stand up or walk due to transitional or permanent disabilities.

There are no contraindications known when using the product as intended.

The maximum user weight is 180 kg.

9 Product Description

Invacare® I-Fit is a shower stool / shower chair composed of four anti-slip height adjustable feet and a plastic seat and backrest (if applicable).

10 Assembling the Product (Fig. 2 — 5)

Installing the Legs (Fig. 2)

CAUTION!

Risk of Trapping

– Take care during assembly.

1. Allow the shower chair to adjust to room temperature, if necessary.
2. Place the seat ① upside down on a flat, stable surface (floor or table).
3. Position one leg ② over one socket ③ in the seat and align.
4. Push the leg down into the socket until it is fully engaged, and the locking tabs ④ latch the openings ⑤ with an audible click.
5. Pull up on the leg to ensure that it has fully locked ⑥ into place.
6. Repeat steps 2-5 for the remaining three legs.

Adjusting Leg Extension Height — Increase Leg Height (Fig. 3)

1. Rotate the leg extension tip ① counterclockwise.
2. Use the number reference ② in the leg extension ③ to adjust the leg height. Refer to detail “B”.
3. Repeat steps 1-2 for the remaining three legs.

CAUTION!

Risk of Falling

– Make sure that all four legs are adjusted to the same height. (Fig. 8)

Adjusting Leg Extension Height — Decrease Leg Height (Fig. 3)

1. Rotate the leg extension tip ① clockwise.
2. Use the number reference ② in the leg extension ③ to adjust the leg height. Refer to detail “B”.
3. Repeat steps 1-2 for the remaining three legs.

CAUTION!

Risk of Falling

– Make sure that all four legs are adjusted to the same height. (Fig. 8)

Installing the Backrest — Model 9781E only (Fig. 4)

1. Before installing the backrest ①, install and adjust the legs so the shower chair is level and stable. See Installing the legs and adjusting leg extension height on Fig. 3.
2. Position the backrest into the slot ② in the seat ③.
3. Push down on the backrest until it snaps securely into the slot in the seat with an audible click.

If both compression buttons ④ are visible, the backrest is attached properly. If the compression buttons are not visible, apply pressure to the backrest until it snaps into the slot in the seat.

Removing the Backrest — Model 9781E only (Fig. 5)

1. Locate and depress both compression buttons ④ with one hand.
2. Using the other hand, use the backrest handle ⑤ to pull the backrest ① up and disengage it out of the slot ② in the seat ③.

11 Maintenance

11.1 General Maintenance Information

This product is maintenance-free, provided that the cleaning and disinfection instructions are adhered to.

11.2 Cleaning and Disinfection

General Safety Information

CAUTION!

Risk of Contamination

– Take precautions for yourself and use appropriate protective equipment.

IMPORTANT!

Wrong fluids or methods can harm or damage the product.

- All cleaning agents and disinfectants used must be effective, compatible with one another and must protect the materials they are used to clean.
- Never use corrosive fluids (alkalines, acid etc.) or abrasive cleaning agents. We recommend an ordinary household cleaning agent such as dishwashing liquid, if not specified otherwise in the cleaning instructions.
- Never use a solvent (cellulose thinner, acetone etc.) that changes the structure of the plastic or dissolves the attached labels.
- Always make sure that the product is completely dried before taking it into use again.

For cleaning and disinfection in clinical or long-term care environments, follow your in-house procedures.

Cleaning Intervals

IMPORTANT!

Regular cleaning and disinfection enhances smooth operation, increases the service life and prevents contamination.

Clean and disinfect the product

- regularly while it is in use,
- before and after any service procedure,
- when it has been in contact with any body fluids,
- before using it for a new user.

Cleaning Instruction

1. Remove the legs and backrest from the shower chair.
2. Use water and a mild, non-abrasive cleaner to clean the shower chair.
3. Dry with a cloth.

Cleaning in a clinical Environment

In addition to cleaning as described above we recommend cleaning the product prior to the use by another patient, after ending of the period of use or on a regular basis (depending on which occurs first) at

- 90 °C for 1 minute
- 85 °C for 3 minutes
- 80 °C for 10 minutes

in the autoclave.

CAUTION!

Risk of burning Skin

– Allow the product to cool after cleaning in the washing machine or autoclave.

Disinfection Instructions

Method: Follow the application notes for the used disinfectant and wipe-disinfect all accessible surfaces.

Disinfectant: Ordinary household disinfectant.

Drying: Allow the product to air-dry.

12 Disposal

Be environmentally responsible and recycle this product through your recycling facility at its end of life.

Disassemble the product and its components, so the different materials can be separated and recycled individually.

The disposal and recycling of used products and packaging must comply with the laws and regulations for waste handling in each country. Contact your local waste management company for information.

13 Reconditioning

This product is suitable for reuse. To recondition the product for a new user, carry out the following actions:

- Inspection
- Cleaning and disinfection

For detailed information, see *11 Maintenance, page 4*.

Make sure that the user manual is handed over with the product.

If any damage or malfunction is detected, do not reuse the product.

14 Technical Data (Fig. 6)

Dimensions			
Ⓐ	500 mm	Ⓓ	415 mm
Ⓑ	375 mm	Ⓔ	440 mm
Ⓒ	375 — 535 mm	Ⓕ	380 — 550 mm (for 9780E shower stool) 745 — 905 mm (for 9781E shower chair)

Weights	
Shower stool I-Fit 9780E	3.7 kg
Shower chair I-Fit 9781E	4.5 kg
Maximum user weight	180 kg

Materials			
①	PP (corpus)	②	TPE (rubber feet)

All product components are corrosion resistant and do not contain natural rubber latex.

15 Environmental Parameters

Operating Conditions	
Ambient temperature	10 - 40 °C
Relative humidity	15% - 93%, non-condensing
Storage and Shipping Conditions	
Ambient temperature	0 - 40 °C
Relative humidity	30% - 75%, non-condensing

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Invacare AG
Invacare Limited
Invacare Australia Pty. Ltd.
Aquatec Operations GmbH