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Cutimed® PROTECT

DE

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Cutimed® PROTECT

EN

Medical skin protection – Foam Applicator

Description of the Product: Cutimed® PROTECT Foam Applicator is a polymeric solution and forms an even film when applied to skin. The product is dispersed in a non-irritating solvent which dries rapidly to form a colourless, transparent film with good permeability to oxygen and water vapour. Cutimed® PROTECT Foam Applicator helps to protect both intact or irritated skin that may arise from urinary and/or faecal incontinence, digestive and wound fluids, adhesives and shearing forces.

Indications: Cutimed® PROTECT Foam Applicator is supplied as a liquid and intended for use as a film-forming product. When applied to intact or irritated skin the product forms a durable, waterproof barrier that works as a protective shield between the skin and bodily waste, fluids, adhesive products and shearing forces. The product is designed to serve as a primary barrier against irritation from bodily fluids.

Contraindications: Cutimed® PROTECT Foam Applicator should not be used as the only skin covering in circumstances that require protection from bacterial contamination or penetration by dressing materials, e.g. catheter sites for intravenous therapy and full or partial thickness wounds. Cutimed® PROTECT Foam Applicator should not be used on infected skin.

Warning: Cutimed® PROTECT Foam Applicator is supplied in a flammable liquid form; the product must be used in well-ventilated areas only. The product should not be used near naked flames or sources of ignition. Keep out of the reach of children. Do not use if the pouch is already open or damaged as the sterility of the foam applicator is guaranteed only when the pouch is unopened and undamaged prior to use. Discard open or unused material. Do not re-use or re-sterilize as there is a risk of transmission of body fluids or contaminated tissue between patients. Do not apply directly into the wound.

Precautions: If skin reddening or any other signs of irritation appear, or if skin reddening or other signs of irritation persist, the use of Cutimed® PROTECT Foam Applicator should be discontinued and a doctor consulted. Use of other barrier products, ointments, creams or lotions may have a significant adverse effect on the efficacy of Cutimed® PROTECT Foam Applicator.

Directions for use:

- (1) Clean and dry the affected area before applying Cutimed® PROTECT Foam Applicator.
- (2 – 3) Remove foam applicator from sachet and apply a uniform coating of film over affected area.
- (4) If an area is missed during the first application, allow Cutimed® PROTECT Foam Applicator to dry (approximately 30 seconds) and then apply again only to the desired area.

When applying Cutimed® PROTECT Foam Applicator to areas with skin folds or areas where skin touches skin, make sure that these areas are sufficiently exposed to air to allow the film to fully dry.

- (5) Using the product under adhesive tape, dressings or devices:
A: allow Cutimed® PROTECT Foam Applicator to fully dry before covering the area with dressing or adhesive products.
B: Cutimed® PROTECT Foam Applicator needs to be re-applied each time the dressing and/or adhesive products is/are changed; most adhesives will remove the barrier film.

Using the product for protection from body fluids, faeces or urine etc., without other adhesive products being applied to the skin:
A: it is recommended to re-apply Cutimed® PROTECT Foam Applicator every 24–96 hours, depending on how often the area is cleaned.
B: in extreme cases (e.g. chronic diarrhoea) which involves very frequent skin cleansing, it may be necessary to re-apply the product more frequently (i.e. every 12–24 hours).

If desired, the film can be removed with most medical adhesive removers following the directions of the manufacturer. Cleanse and dry the area and then re-apply Cutimed® PROTECT Foam Applicator.

Cutimed® PROTECT Foam Applicator shall be applied by healthcare professionals only.

Ingredients: Disiloxane, Acrylate Copolymer

Storage and Disposal: Cutimed® PROTECT Foam Applicator should be stored in dry conditions. Do not expose to direct sunlight. Store below 25°C/77°F. (6) Disposal should be made according to local regulations.

Notice: Any serious incident that has occurred in relation to this device should be reported to BSN medical GmbH and the competent authority of your state.

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© = registered trademark