JOBST® LymphCARE

DE

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AU

Bandage set for oedema management

Mode of application:
The dressing method shown on the pictures is a recommendation. All products contained in the (JOBST® LymphCARE System) should be used according to their indications and contraindications (see instructions for use). Cautions have to be followed in accordance with the instructions for use. Tricofix® elastic tubular bandage for skin protection is applied first. Elastomull® elastic fixation bandage is used for bandaging fingers and toes. Artiflex® synthetic bandage is used as padding material on top of Tricofix®. Comprilan® textile elastic short stretch bandage is applied over Artiflex®. Leukoplast® hospital fixes the ends of the Comprilan® bandages.

Other recommendations:
Incorrectly applied bandages may cause a problem during wearing. Open wounds must be covered with an appropriate dressing before the bandages are put on.
Artiflex®: 40% polyester, 40% polypropylene, 20% polyethylene
Comprilan®: 100% cotton
Elastomull®: 42% cotton, 29% viscose, 29% polyamide
Leukoplast® hospital: Viscose fabric coated with rubber-based adhesive.
Tricofix®: 100% cotton
Leukoplast® hospital contains dry natural rubber. Contains colophony.

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