*The natural way of building.*

# SISSEL® PUTTY

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**General information:** read the following user manual carefully and follow the safety instructions and warnings. The SISSEL® Putty is a therapy clay for numerous simple and safe therapeutic finger and hand exercises. **Safety instructions:** ! Check the usability of the product before each use. Do not use the product if it is heavily contaminated or if foreign bodies are included. ! Choose an exercise program that suits your health level.

and avoid overly difficult exercises or experiments with increased risk. ! Use only accessories intended by the manufacturer. ! For external use only! ! The product is not suitable for consumption. ! The product is not a toy and should therefore be kept out of the reach of children. Do not allow children to play with

it. There is a risk of swallowing and choking! **Product information:**  
 Scope of delivery: 1x Putty, 1x user manual **Technical specifications:** Material: Silicone elastomer (hypoallergenic, latex-free, oil-

free, odorless) Weight: approx. 85g **Intended use:** Stop exercising if you experience pain or swelling. If pain or swelling persists after use consult a physician or therapist. ⚠️ Never heat the SISSEL® Putty (water bath, microwave, oven, grill). ⚠️ Use the product only according to therapeutic instructions. Instruction and supervi-

tion by qualified personnel is recommended. ⚠ Risk of injury! The therapy clay is for external use only and must not be swallowed. Contact with eyes, nose, mouth and ears should be avoided. Contact with injured or inflamed skin should

should be avoided. Contact with injured or inflamed skin should be avoided. **Indication:** Finger mobility disorder, contractures of the finger joints, strengthening and endurance of the thumb, finger and hand muscles, strengthening and endurance of the forearm and grip muscles, improvement of coordination, voluntary movements, tactile sensitivity, tactile perception, sport-related increase of grip strength and

sensitivity, tactile perception, sport-related increase of grip strength and endurance. **Contra Indication:** Acute injuries (open wounds, soft tissue injuries, fractures), acute radiating pain, unexplained disease findings, managing compliance (anxiety, disgust, pain). **Regulatory require-**

**ments:** The present product of SISSEL® GmbH complies with the following regulatory requirements: Regulation (EU) 2017/745 on medical devices. All serious incidents occurring in con-

In connection with the product must be reported immediately to the manufacturer and the competent authority. **Usage:** Check the packaging and the product for any contamination upon deli-

very.  Never use a contaminated product. Exercise according to exercise instructions or as directed by therapeutic professionals. **Care & Hygiene:** Single patient multiple use. To protect against drying out, the therapy clay should be stored in the accompanying resealable cur

the therapy clay should be stored in the accompanying resealable cup after use. Hand disinfectants that can damage the materials of the product should be avoided. The application instructions of the disinfectant manufacturer must be observed. **Disposal & Environment:**

At the end of use, the product must be disposed of in accordance with local regulations. **Warranty:** The warranty is governed by the legal regulations.

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SISSEL® GmbH, Bruchstrasse 48,