The natural way

SISSEL[®] SOFT CURVE

- | Gebrauchsanweisung
- | Instructions for use
- | Mode d'emploi
- | Gebruiksaanwijzing
- | Instrucciones de uso
- | Istruzioni per l'uso

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General information

Read the following user manual carefully and observe the safety instructions and warnings. The SISSEL® Soft Curve is a pillow that gently supports the cervical spine thanks to its special shape and thus relieves pressure on the entire shoulder and neck area. The product may only be used in accordance with these instructions for use. SISSEL® GmbH accepts no liability for the use of the product outside the applications described in these user instructions. The user manual must be kept carefully and within easy reach, and must be enclosed if the product is passed on.

Safety instructions

- ! Check the condition of the product before each use. If damaged, the function of the product and safety is no longer guaranteed.
- ! Do not use the product in the immediate area of heat sources (e.g. radiator, hotplate, open flame, flying sparks) and avoid direct sunlight.
- ! Keep the area of use free of sharp or pointed objects.
- ! Use only accessories recommended by the manufacturer.
- ! In case of uncertainty or existing health problems in the shoulder neck area or cervical spine, use the product only after consulting a doctor.

Product information

Scope of delivery

1 x SISSEL® Soft Curve, 1 x user manual, 1 x cover

Technical data

available in four different sizes:

Size S

Dimensions (L x W x H): approx. 63 x 30 x 8 cm

Weight: approx. 1,14 kg

Size M

Dimensions (L x W x H): approx. 63 x 32 x 10 cm

Weight: approx. 1,59 kg

Size L

Dimensions (L x W x H): approx. 63 x 32 x 13 cm

Weight: approx. 1,96 kg

Compact

Dimensions (L x W x H): approx. 48 x 32 x 10 cm

Weight: approx. 1,33 kg

Material Filling: Viscoelastic foam

Material Cover: Velour (75 % cotton, 25 % polyester)

 **Risk of injury!** Do not use the product in case of lack of compliance (due to fear or pain), acute injuries and/or sprains of the cervical spine, acute cervical spine syndrome (cervical syndrome), acute cervical vertigo (dizziness), acute radiating pain, abnormal sensations, movement disorders, acute stroke, acute slipped disc of the cervical spine, acute inflammatory processes such as meningitis and encephalitis, acute or chronic nerve root inflammation of the cervical spine, spastic torticollis (torticollis spasmodicus), neurological diseases or malformations, tumour diseases only after prior consultation with a doctor.

Accessory

The product may only be used with the accessories supplied.

Usage

1. Check the packaging and the product for any damage upon delivery.

 Never use a damaged product.

2. Check the product for integrity and contamination before each use.

3. Use the product as intended.

4. Use the SISSEL® Soft Curve instead of your usual pillow for sleeping. The pillow is designed especially for side sleepers, but also for back sleepers.

The special shape positions the cervical spine in an anatomically correct, physiological position and relieves the neck muscles and intervertebral discs. The SISSEL® neck pillow therefore makes a significant contribution to relaxing the neck muscles and thus to promoting deep, healthy, restful sleep.

5. Lie on your side with your head on the SISSEL® Soft Curve so that the high side is against your shoulder and the cervical spine is supported by the cushion. The SISSEL® Soft Curve is correctly positioned when the high side fills the area where the cervical spine meets the shoulder.

Note: When using the SISSEL® Soft Curve, the new sleeping position may initially cause a change in sleeping habits for a few nights. As a rule, this adjustment lasts a few days. Only in exceptional cases can it take longer. Cleaning measures may be necessary after use (see Care & Hygiene).

Care & Hygiene

The viscoelastic foam core can be cleaned by hand using a mild detergent and lukewarm water. Carefully squeeze it out by hand, never wring it out! Then leave to air dry on a rack (place a towel underneath) for a sufficient length of time (approx. 1 week). Do not dry in the sun or on a radiator. Discoloration of the pillow core is possible and does not affect the function of the product. The velour cover is machine washable at 40 °C. Then air dry. Follow the washing label.

Note: The SISSEL® Soft Curve cannot be disinfected.

Storage: For storage, protect the product from direct sunlight and store it in a clean, dark and dry place when not in use.

Disposal & Environment

The product and the accessories supplied with it can be disposed of with household waste. No special types of packaging are used. The respective valid disposal guidelines must be obeyed.

Warranty

The warranty is governed by the statutory provisions. Any warranty is excluded if goods delivered by SISSEL® GmbH have been processed, treated or modified by others without consent or if the instructions for use have not been followed.

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SISSEL GmbH