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
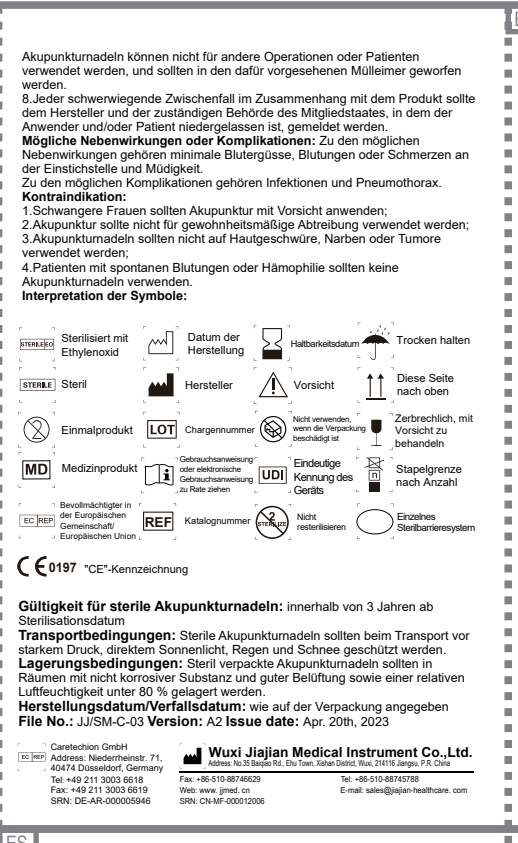
Medical Device Operation Manual

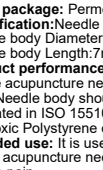CE 0197

Product name: Sterile Acupuncture Needles

Sterilization method: EO sterilization

Initial package: Needle package + PVC. Permeable paper + PE film

Specification: Needle body diameter: Needle body Length
Needle body diameter: 0.13mm-0.15mm, needle interval 0.01mm
Needle body Length: 7mm-150mm, length interval 1mm

Product performance and structure:
Sterile acupuncture needles are made of needle handle and needle body. Needle body should be made of S316L18-10 Stainless steel wire stoipic in ISO 15510:2010 standard. Material of needle handle is non-toxic Polystyrene or Conductive plastic.

Intended use: It is used by the professional acupuncturists, who use the sterile acupuncture needles on the human body to stimulate the body to relieve pain.

Indications: Sterile Acupuncture Needles are used in acupuncture therapy to relieve pain: Primary osteoarthritis of knee, Fibromyalgia syndrome, Cervical spine pain, Pain in joint, Persistent headache associated with traumatic injury to the head and Intervertebral disc stenosis of neural canal, lower extremity.

Intended patient population: Adults with no contraindications.

Intended users: Professional acupuncturists.

Product operation instruction:

1. Read operation manual before use.
2. Disinfect acupoints in the affected part with alcohol cotton. Keep skin dry and clean.
3. Tear open Permeable paper on needle handle. Hold it with finger. Take out sterile acupuncture needle from the package.
4. Hold needle handle with finger, with needle tip pointing towards acupoint. Insert tip to the skin with acupuncture area. (If the acupuncture needles are with plastic pipe, professional acupuncturists can decide whether to use it for needle positioning or not. If using the plastic pipe, hold the needle handle with the thumb and the index finger of the other hand, insert the needle into the skin, then remove the plastic pipe.)
5. Keep pressing needle handle with finger until the tip goes to a proper depth.

Precautions and Warnings:

1. As a medical device, it must be used by qualified physician.
2. Do not use when the package is damaged.
3. Do not reuse because it is single-use.
4. Use it within the validity time.
5. Those who are allergic to the material of the needle body should use with caution or follow the instruction of an acupuncture physician
6. Please throw used acupuncture needles into designated dustbin.
7. After the sterile barrier system is opened, the sterile acupuncture needles should be used as soon as possible. The unused sterile acupuncture needles cannot be used for other operations or patients, and should be discarded into designated trash bins.
8. Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

Product name: Sterile Acupuncture Needles

Sterilization method: EO

Initial package: Permeable paper + PVC. Permeable paper + PE film

Specification: Needle body Diameter: Needle body Length
Needle body Diameter: $\Phi 0.13\text{mm}-\Phi 1.00\text{mm}$, diameter interval 0.01mm
Needle body Length: 7mm, 13mm, 16mm, 19mm, 25mm
Needle body Length interval 1mm

Product performance and structure:
Sterile acupuncture needles are made of needle handle and needle body. Needle body should be made of X5CrNi18-10 Stainless steel wire stipulated in ISO 15570:2010 standard. Material of needle handle is non-toxic Polystyrene or Conductive plastic.

Intended use: It is used by the professional acupuncturists, who use the sterile acupuncture needles on the human body to stimulate the body to relieve pain.

Indications: Sterile Acupuncture Needles are used in acupuncture therapy to relieve pain: Primary osteoarthritis of knee, Fibromyalgia syndrome, Cervical spine pain, Pain in joint, Persistent headache associated with traumatic injury to the head and Intervertebral disc stenosis of neural canal, lower extremity.

Intended patient population: Adults with no contraindications.

Intended users: Professional acupuncturists.

Product operation instruction:
1. Read operation manual before use.
2. Disinfect puncture sites in the affected part with alcohol cotton. Keep skin dry and clean.
3. Tear open Permeable paper on needle handle. Hold it with finger. Take out sterile acupuncture needle from blister.
4. Hold needle handle with finger, with the tip pointing towards acupoint. Insert tip to the skin with acupuncture area. (If the acupuncture needles are with plastic pipe, professional acupuncturists can decide whether to use it for needle positioning or not. If using the plastic pipe, hold it with one hand and hold the needle handle with the fingers of the other hand. Insert the needle into the skin and then remove the plastic pipe.)
5. Keep pressing needle handle with finger until the tip goes to a proper depth.

Precautions and Warnings:
1. Hold needle handle with finger and must be used by qualified physician.
2. Do not use when the package is damaged.
3. Do not reuse because it is single-use.
4. Use it within the validity time.
5. Those who are allergic to the material of the needle body should use with caution or follow the instruction of an acupuncture physician
6. Please throw used acupuncture needles into designated dustbin.
7. After the sterile barrier system is opened, the sterile acupuncture needles should be used up at one time, the unused sterile acupuncture needles should not be used on other patients or operators or patients, and should be discarded into designated trash bins.
8. Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

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Cartechong GmbH
Wuxi Jiajian Medical Instrument Co., Ltd.