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en Sterifix® Filter Needle

with 5 µm Filter – Luer
415030 – 19 Ga. x 1-½ in. (3,8 cm)
415040 – 19 Ga. x 1 in. (2,5 cm)

Sterile filter needle with a particulate matter filter, when attached to a syringe, is used for solution withdrawal or injection. The filter is intended to filter particulates equal to or greater than 5 µm.

TARGET PATIENT GROUP: Any patient population; for use when solution to be administered requires filtration at either admixture or administration.

INTENDED USER: The device is used by trained clinicians as prescribed by a physician.

WARNING: RE-USE OF SINGLE-USE DEVICES CREATES A POTENTIAL RISK OF PATIENT OR USER. IT MAY LEAD TO CONTAMINATION AND/OR IMPAIRMENT OF FUNCTIONAL CAPABILITY. CONTAMINATION AND/OR LIMITED FUNCTIONALITY OF THE DEVICE MAY LEAD TO INJURY, ILLNESS OR DEATH OF THE PATIENT.

CAUTION: DO NOT ASPIRATE AND INJECT USING SAME NEEDLE.

DISPOSAL: Dispose of according to local guidelines and/or clinical protocol.

NOTICE TO THE USER: All serious incidents related to this product must be reported to B. Braun and the competent authority in the country where the product is being used.

INSTRUCTIONS FOR USE: Use Aseptic Technique.

- Visually inspect sterile barrier system for breaches and integrity prior to use. Do not use if packaging or product is damaged or contaminated or if protective caps are loose or missing. Use of damaged or contaminated device might lead to injury, illness or death of the patient.

FOR ASPIRATION:

- Grasp needle guard and firmly attach filter needle to luer fitting of syringe. The plastic needle guard serves as a wrench to attach and remove the filter needle.
- Cleanse site before insertion. Remove needle guard and insert needle into rubber stopper of fluid container.
- Withdraw fluid through the filter needle into syringe.
- Remove filter needle before injection.

FOR INJECTION: Firmly attach hub to luer fitting of filled syringe. Cleanse site and inject into injection site or rubber stoppered vial.

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