en **Instructions for use/Technical description**

Disposable vein extirpation set

USA

Note for U.S. users

This Instructions for Use is NOT intended for United States users. Please discard. The Instructions for Use for United States users can be obtained by visiting our website at www.aesculapusaifus.com. If you wish to obtain a paper copy of the Instructions for Use, you may request one by contacting your local Aesculap representative or Aesculap's customer service at 1-800-282-9000. A paper copy will be provided to you upon request at no additional cost.

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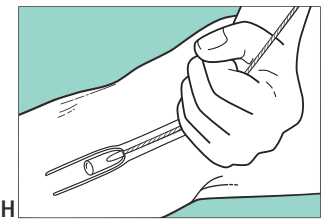
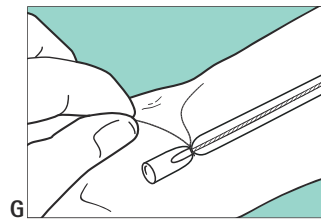
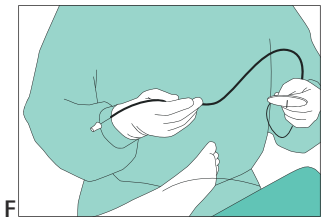
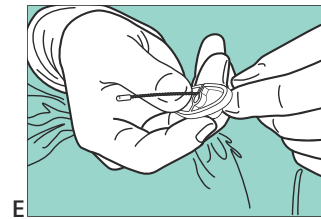
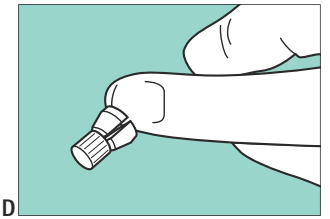
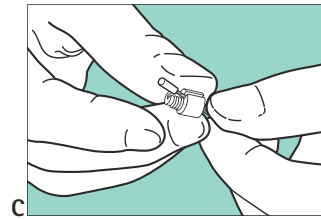
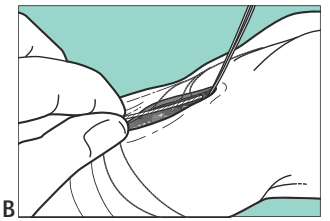
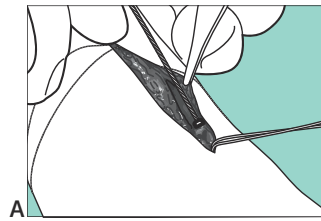
12158135 2025-04 Change No. AE0064639

Legend

- A Insertion of Venostrip into the great saphenous vein
- B Inserted Venostrip in the great saphenous vein
- C Fixing of the stripper head on the guidewire
- D Screw cap fixed on stripper head
- E Insertion of the guidewire into the handle
- F Visual control of the completely extirpated great saphenous vein
- G Fixation of the vein below the fitting of the stripper wire
- H Inverting the vein over the stripper by pulling the wire

1 About this document

Note

General risk factors associated with surgical procedures are not described in this document.

1.1 Scope

These instructions for use apply to the following products:

Art. no.	Designation	Packaging unit
5523184	Complete set, sterile	Carton of 10 pieces
5523176	Stripper wire, sterile	Carton of 10 pieces

Note

Instructions for use and further information about B. Braun/AESCULAP products can be found on the B. Braun eFU website at eifu.bbraun.com

1.2 Safety messages

Safety messages make clear the dangers to patient, user and/or product that could arise during the use of the product. Safety messages are labeled as follows:

⚠ WARNING

Indicates a possible threat of danger. If not avoided, minor or moderate injury may result.

2 Clinical use

2.1 Product description

Venostrip, a disposable vein stripper according to Prof. Dr. Grub, consists of a polymer-coated guidewire with fittings at each end of the wire, a handle and 3 stripping heads in 9 mm, 12 mm, and 15 mm diameter and one screw cap to fasten the stripping heads.

To adapt to both surgical procedures, endostripping and exostripping, Venostrip is available as a complete set (5523184) or with the wire only (5523176). The set components can be optionally connected to both wires, either wire 5523176 or the wire within 5523184.

2.2 Components required for use

The stripper wire 5523176 can be either used as a stand-alone device (endostripping) or as a spare part for the wire within set 5523184 (endostripping and exostripping). The set components (stripping heads, screw cap and handle) can be combined individually to both wires, either wire 5523176 or the wire within set 5523184, depending on which procedure is performed.

2.3 Areas of use and limitations of use

2.3.1 Intended purpose

For Intended Purpose, see Intended use.

2.3.2 Intended use

Venostrip can be used for both exostripping and endostripping.

2.3.3 Clinical benefit

Higher quality of life compared to Radiofrequency ablation (RFA) or Cyanoacrylate embolization (CAE) intervention, as reported by Gasior et al. (2022), DOI: 10.1097/SLA.0000000000004914.

2.3.4 Indications

Note

The manufacturer is not responsible for any use of the product against the specified indications or the described applications.

Extirpation of the great and small saphenous vein or parva vein in primary varicosis, chronic venous insufficiency and in the case of serious cosmetic varicosis.

2.3.5 Absolute contraindications

Do not use Venostrip in case of acute inflammation, e.g., acute thrombophlebitis, florid erysipelas or acute phlebothrombosis.

2.3.6 Relative contraindications

The following conditions, individual or combined, can lead to higher complication rates:

- patients with heart insufficiency or lung diseases
- pregnancy and thereby related hormonal changes
- elderly patients

In the presence of relative contraindications, the user decides individually regarding the use of the product.

2.3.7 Intended user group

The product is only used by professionals who possess a medical degree and additionally passed a specialization education in vascular surgery.

2.3.8 Intended patient population

There is no restriction regarding the intended patient population in addition to the indications/contraindications.

2.4 Risks, adverse effects and interactions

As part of the legal duty to inform, the following typical risks, interactions and side effects associated with the use of surgical instruments are pointed out.

Possible risks, interactions and side effects of the application currently known to the manufacturer are:

- Haematomas around the extirpated vein and their side branches.
- Infections
- Thrombus formation
- Nerve and lymphatic vessel injuries
- Oedema
- Deep leg vein thrombosis
- Lung embolism

2.5 Safety information

2.5.1 Clinical user

General safety information

To prevent damage caused by improper setup or use and to not compromise the manufacturer warranty and liability:

- ▶ Use the product only according to these instructions for use.
- ▶ Follow the safety and maintenance instructions.
- ▶ Ensure that the product and its accessories are only operated and used by qualified personnel.
- ▶ Store any new or unused products in a dry, clean and safe place.
- ▶ Prior to use, check that the product is in good working order.
- ▶ Keep the instructions for use accessible for the user.

Note

The user is obligated to report all severe events in connection with the product to the manufacturer and the responsible authorities of the state in which the user is located.

Notes on surgical procedures

It is the user's responsibility to ensure that the surgical procedure is performed correctly.

Appropriate clinical training as well as a theoretical and practical proficiency of all the required operating techniques, including the use of this product, are prerequisites for the successful use of this product.

The user is required to obtain information from the manufacturer if there is an unclear preoperative situation regarding the use of the product.

2.5.2 Product-specific safety information

The indication for surgery in varicosis requires great responsibility.

- ▶ Check the clotting factors before surgery for varices.
- ▶ If required, perform additional investigative procedures such as venous occlusion phletysmography and phlebodynamometry.
- ▶ Exercise great caution and a special incision in varicotomy in patients with attendant lymphoedema.
- ▶ In all cases ensure the functional integrity of the deep veins by suitable investigations such as Doppler ultrasonography, duplex scanning or ascending phlebography before use.

2.5.3 Sterility

The product has been EO-sterilized (ethylene oxide) and is supplied in sterile packaging.

The safe sterile provision of the product is only guaranteed if the sterile packaging is undamaged and unopened and the use-by date has not passed.

- ▶ Store products in their original packaging in a dust-protected, dry and temperature-controlled room.
- ▶ Remove products from their original protective packaging only just prior to application.
- ▶ Visually inspect the product packaging to ensure that the sterile barrier system is intact.
- ▶ Do not use products from open or damaged sterile packaging.
- ▶ Do not use the product after its use-by date.

The product is intended for single-use.

- ▶ Do not reuse the product.

The processing of the product affects its functionality. Risk of injury, illness or death due to soiling and/or impaired functionality of the product.

- ▶ Do not process the product.

2.6 Application

⚠ WARNING

Risk of injury and/or malfunction!

- ▶ Prior to each use, inspect the product for loose, bent, broken, cracked, worn or fractured components.

Depending on the location the diseased trunk vein is surgically severed in the patient's groin, at the back of the knee and/or at the ankle and then pulled out with a wire. Thereby the vein is either directly dragged by stripper heads (exostripping) or invaginated and pulled out from the inside to the outside (endostripping).

The following described procedure is one example for the complete stripping of the great saphenous vein. Variations of the procedure (including the extirpation of other indicated veins) are possible and are subject to the decision of the health care specialist.

- ▶ Carefully dissect the great saphenous vein through a cutaneous incision running from above the inguinal fold to the junction with the femoral vein.
- ▶ Tie off all the high tributaries with ligatures.
- ▶ Double ligate the great saphenous vein at the level of the junction.
- ▶ Locate the distal end of the great saphenous vein through a second incision over the first metatarsal.
- ▶ Divide and ligate the dorsal veins individually.

Note

In principle the stripper may be introduced in either direction (valve incompetence).

- ▶ However, always first attempt to advance the stripper proximally from the foot to the groin, see Fig. A & Fig. B.
- ▶ Make a transverse incision in the vein over the groin and retrieve the stripper.
- ▶ Secure the stripper with a strong ligature.
- ▶ Then complete division of the vein below the junction with the deep vein and screw a stripper head of a suitable size onto the stripper, see Fig. C & Fig. D.
- ▶ If needed, clip the stripper handle onto the distal end of the stripper wire, see Fig. E.
- ▶ With the stripper in situ, extirpate the local lateral branches and previously marked convoluted varices.
- ▶ Locate perforating veins by Doppler ultrasonography or phlebography and divide through short separate incisions.
- ▶ Finally extract the great saphenous vein, see Fig. F, always from proximal to distal to avoid lesions of the saphenous and sural nerves.

– or –

- ▶ If endostripping is preferred to exostripping, secure the vein with a strong ligature below the fittings of the stripper wire, see Fig. G, and pull gently on the wire or handle, to invert the vein over the stripper, see Fig. H.
- ▶ The remainder of the procedure follows the description given above.
- ▶ Stripping heads and the screw cap are not needed to perform endostripping.
- ▶ The handle can be attached optionally.

3 Disposal

⚠ WARNING

Risk of infection due to contaminated products!

- ▶ Adhere to national regulations when disposing of or recycling the product, its components and its packaging.

⚠ WARNING

Risk of injury due to sharp-edged and/or pointed products!

- ▶ Ensure that the packaging prevents injury by the product when disposing of or recycling the product.

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