**• HEINE® XP Disposable Laryngoscope  
Handle Shell with bottom insert**

CE

**HEINE Optotechnik GmbH & Co. KG**  
Kientalstr. 7 · 82211 Herrsching · Germany  
Tel. +49 (0)8152/38-0  
Fax +49 (0)8152/38-202  
E-Mail: info@heine.com · [www.heine.com](http://www.heine.com)  
med 0615      2015-05-06

# **HEINE® XP Disposable Laryngoscope Handle Shell with bottom insert**

<b>DEUTSCH</b>	<b>3</b>
<b>ENGLISH</b>	<b>5</b>
<b>FRANÇAIS</b>	<b>7</b>
<b>ESPAÑOL</b>	<b>9</b>
<b>ITALIANO</b>	<b>11</b>
<b>SVENSKA</b>	<b>13</b>
<b>NEDERLANDS</b>	<b>15</b>
<b>DANSK</b>	<b>17</b>
<b>SUOMI</b>	<b>19</b>
<b>PORTUGUÊS</b>	<b>21</b>

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## Handle Shell with bottom insert

ENGLISH

 Please read and follow these instructions for use of and keep them for future reference.

### Intended Use

This XP handle shell is designed for single use only for endo-tracheal intubation with a fiber optics laryngoscope blade or XP blade and will only function when fitted with a HEINE Standard (XHL/LED) laryngoscope inserts including bottom insert.

**For U.S. only:**

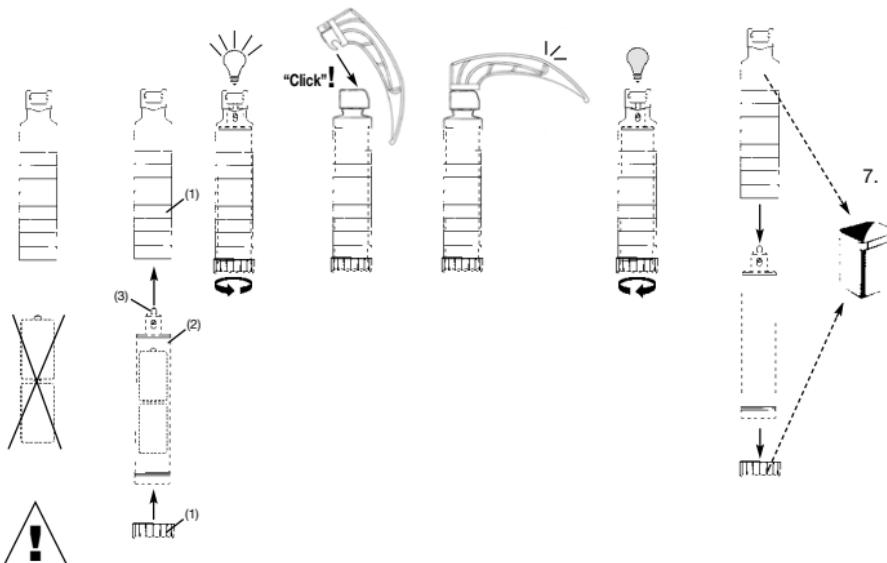
Federal law restricts this device to sale by or on the order of a Physician or Practitioner!

### Warnings and Safety Information

-  **CAUTION!** Indicates potential hazardous situations. Ignoring the corresponding instructions may lead to dangerous situations of mild to moderate extent. (Background color yellow; foreground color black).
-  **NOTE!** Indicates valuable advice in terms of installation, operation, maintenance or repair. Notes are important, but not related to hazardous situations.

### Setting up

1. 2. 4. 5. 3. 6.

### Operation

We can only guarantee the function of this unit if a HEINE Standard (XHL/LED) laryngoscope insert (2), a HEINE bulb (3) and HEINE spare parts are used.

Bring the blade into the normal working position and check that the light switches on.

The bulb brightness can reduce during use if the rechargeable battery was not charged sufficiently before use. We recommend a minimum recharge time of 10 minutes for HEINE rechargeable batteries in HEINE chargers.

After some minutes' use the laryngoscope bulb will be hot.

The laryngoscope may only be used by trained personnel.

## **General Warnings**

-  Check the correct operation of the device before use! Do not use the device if there are visible signs of damage or the light begins to flash.
- Do not use the device in fire- or explosive risk area (e.g. oxygen saturated or anaesthetic environments)
- Do not modify the device.
- Use only original HEINE parts, spare parts, accessories and power sources.
- The setup and operation of the HEINE Standard (XHL/LED) laryngoscope insert are described in a separate instruction for use.

## **General Notes**

-  The warranty for the entire product is invalidated if non-genuine HEINE products or non-original parts are used and if repairs or modifications are made to the device by persons not authorized by HEINE. For more information, please visit [www.heine.com](http://www.heine.com).
- Only plastic- and metal blades to the green standard (ISO 7376-3) may be used.
- Cross-contamination can only be avoided if a non-contaminated battery insert is used.

The appendix contains following tables:

- Technical specification
- Explanation of the used symbols

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